Manual de Usuario de Vid2DICOM

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Pantalla Principal


The screenshot shows the main interface of the Vid2DICOM application. The window title is "Vid2DICOM - 1.0.5". The interface includes a form for entering patient and study information. The fields are: Patient ID (empty), Patient Name (empty), Patient BD (5/ 7/2012), Patient Sex (Male), Accession Number (empty), and Study Description (empty). There are four buttons: "Search PACS", "Search WL", "Setup", and "Clear". Below the form is a table with the following columns: Name, ID, Sex, Birth Date, Study Description, Accession N., and Study Inst. The table has one row highlighted in blue. At the bottom of the window, there are two buttons: "Acquire" and "End", and a copyright notice: "(c) www.CharruaSoft.com".

Name	ID	Sex	Birth Date	Study Description	Accession N.	Study Inst.

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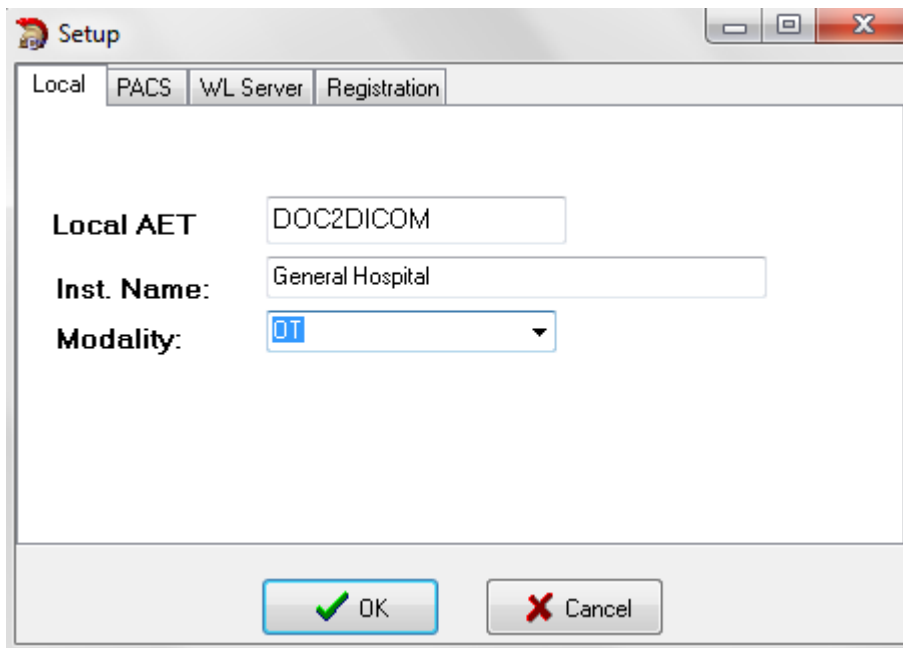
The screenshot shows a 'Setup' dialog box with the 'Local' tab selected. The fields are:

- Local AET:** DOC2DICOM
- Inst. Name:** General Hospital
- Modality:** OT

Buttons for 'OK' and 'Cancel' are located at the bottom of the dialog.

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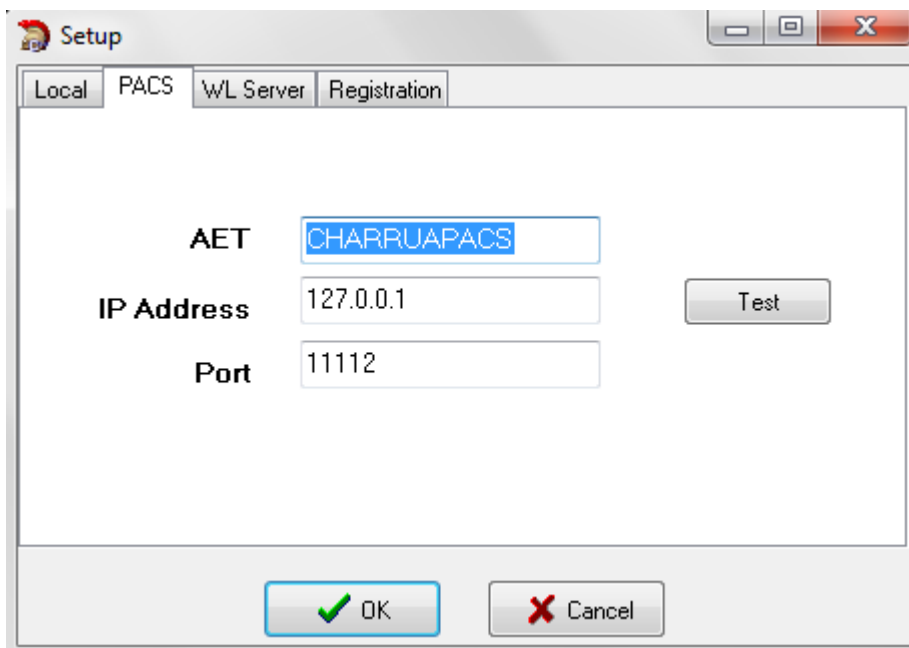The screenshot shows the 'Setup' dialog box with the 'PACS' tab selected. The fields are:

- AET:** CHARRUAPACS
- IP Address:** 127.0.0.1
- Port:** 11112

A 'Test' button is located to the right of the IP Address field. Buttons for 'OK' and 'Cancel' are located at the bottom of the dialog.

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